



PHILIPS
RESPIRONICS

DreamWear

Under the nose
nasal mask

Rediscover
dreams

©2015 Koninklijke Philips N.V. All rights reserved.
Specifications are subject to change without notice.
Trademarks are the property of Koninklijke Philips N.V.
(Royal Philips) or their respective owners.

Broudy JS 10/12/15 MCI 4106983 PN 1125882

Philips Respironics
1010 Murry Ridge Lane, Murrysville, PA 15668 USA

www.philips.com.au

Like wearing nothing at all

Patients can sleep how they want with the freedom and comfort of DreamWear

Patients can sleep like it's not even there

Do your patients have trouble finding a comfortable sleeping position or feel restricted by CPAP tubing? DreamWear is an innovative mask created to give your patients the choice to sleep how they want.

The best of pillows and nasal masks

DreamWear's unique design offers many of the benefits of nasal and pillows masks to allow patients to have the best of both mask types. Allowing more freedom of movement and more comfort than their prescribed mask, DreamWear patients feel like they are not wearing a mask at all.



Benefits

- Tubing attachment located at the top of the head to offer freedom of movement during the night
- Prevents red marks, discomfort or irritation in the nostrils or on the nose bridge**
- Users rated DreamWear more stable, more comfortable, and more appealing than their prescribed mask*
- Soft, flexible silicone frame and fabric wraps provide a soft touch on the patient's cheek

Users told us that DreamWear gave them more freedom to move, had a better fit than their prescribed mask, and comes closest to making them feel like they don't have anything on their face during therapy.*



DreamWear cushion size is determined by measuring the length and width of the nose using the simple sizing gauge.***

*2015 Philips User Preference Questionnaire, data on file.

**Mask does not directly contact the bridge of the nose or nostrils

***Medium wide (MW) cushion size uses length and width of the nose

Get started

To learn more about why DreamWear can be one of the only nasal masks you offer your patients, contact your Philips Respironics representative for more information.

DreamWear fitpak

Part number

DreamWear with headgear, medium frame,
all cushion sizes

1116700

DreamWear replacement parts

Part number

Small nasal cushion

1116740

Medium nasal cushion

1116741

Large nasal cushion

1116742

Medium wide nasal cushion

1116743

Headgear

1116750

Small frame

1116745

Medium frame

1116746

Large frame

1116747

Fabric wraps

1116754



DreamWear patient fitting guide

Achieving the right fit with DreamWear is easy. Simply follow the instructions below to get started.

Four cushion sizes (Small, medium, large, medium wide) are available. Please read instructions for use for further fitting information.

- 

1 Hold the sizing gauge horizontally under the nose and select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.
- 

2 The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.
(At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.)
- 

3 With the mask assembled, place the cushion under the nose.
- 

4 Position the frame on the top of the head.
- 

5 Pull the headgear over the back of the head.
- 

6 To adjust the mask, peel the headgear tabs away from the fabric. Adjust the straps length. Press the tabs back against the fabric to reattach.
(Note: Do not overtighten the headgear.)

- 

7 Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

Mask frame sizing

The medium (MED) mask frame will comfortably fit most faces. If the MED frame does not fit a patient's face, a small (SM) or large (LG) mask frame might better suit your patient's needs.

Part of the Dream Family

DreamWear is part of the Dream Family from Philips Respironics. The Dream Family offers innovative, comprehensive sleep therapy technology like our sleep therapy system, DreamStation, and our patient self-management tool, DreamMapper.



Call for more information, 1300 766 488.